

# touchControl smartphone app

The touchControl™ App provides you with a convenient way to control your hearing aids. The app is available for Apple® and Android™ mobile devices. It can be downloaded free of charge from the Apple App or Google Play™ stores.



The app uses very high pitched sounds that you cannot hear to control your hearing aids. The app may ask you to turn the volume of the phone up or down to better communicate with your hearing aids.

## App download.

App-solutely easy to get!

Easy to use and easy to get: Simply download the touchControl App to your Android or iOS smartphone on the Google Play Store or Apple App Store. The best part is the download is free!

Step 1




Make sure you have a Google Play Store account for your Android phone or an Apple App Store account for your iPhone.

Step 2



Log into your Google Play Store or Apple App Store account with your Android or iOS smartphone. Find the touchControl App.

Step 3

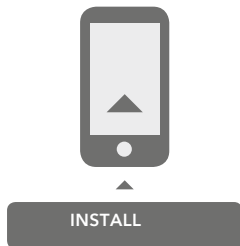


Follow the instructions to install the app on your smartphone.

Step 4



Discover how easy it is to control your hearing instruments by simply trying your new app.

## touchControl Home Screen

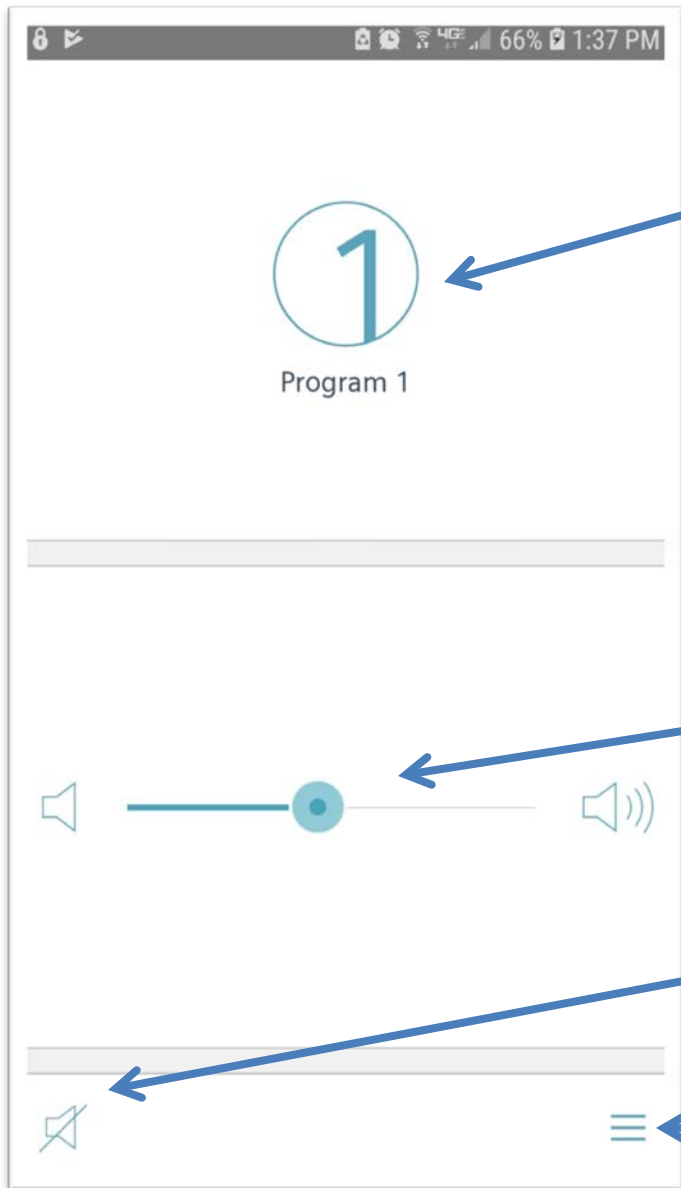


Change the program by tapping on the top portion of the screen

Adjust the volume using the slider

Mute and unmute your hearing aids

Settings

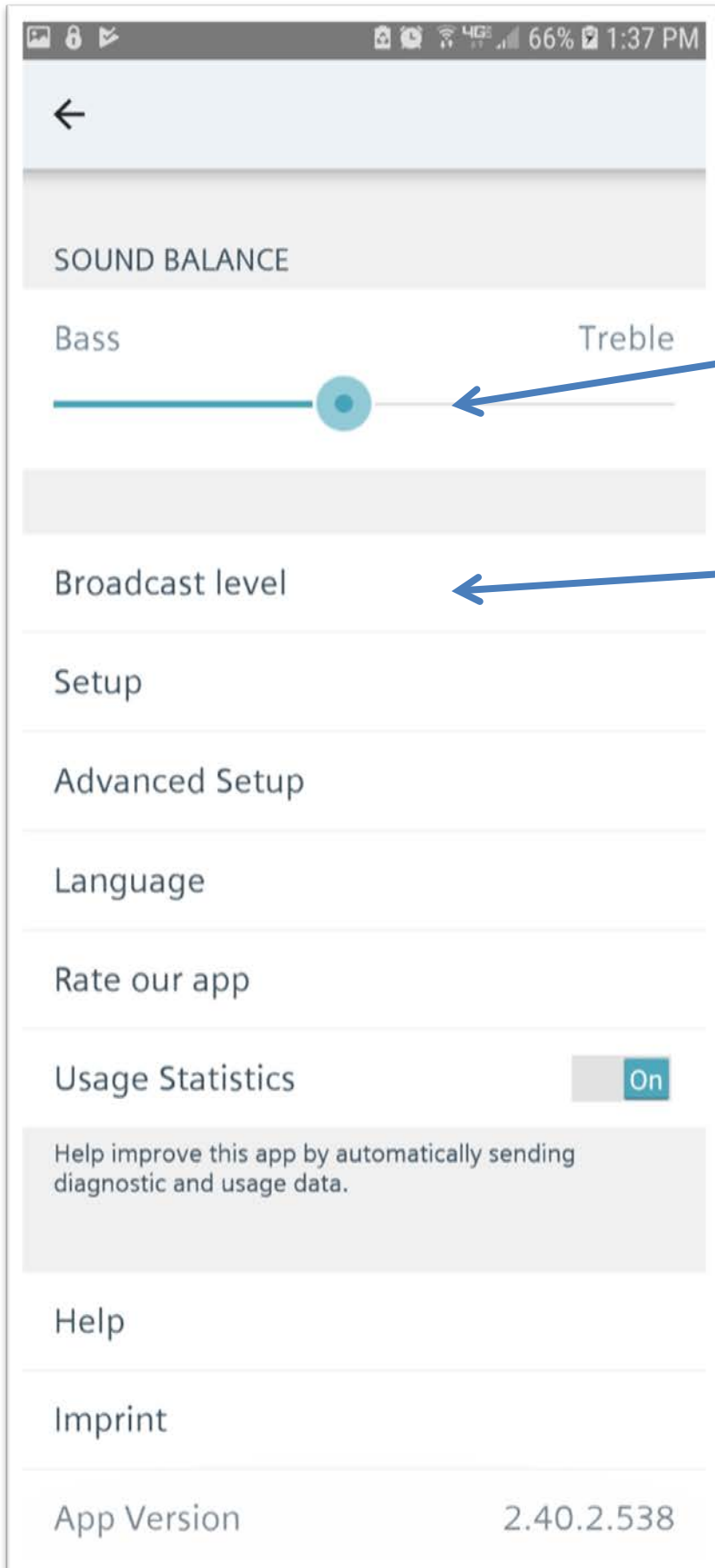

## Settings Screen

This feature is not active with the Exact hearing aids.

Broadcast Level controls the strength of the ultrahigh pitched control signal from the app to the hearing aids. Adjust the broadcast level if the hearing aids are not as responsive as they should be or if the control signal is too loud.