


Exact Quick Start Guide

Parts of the Hearing Aid


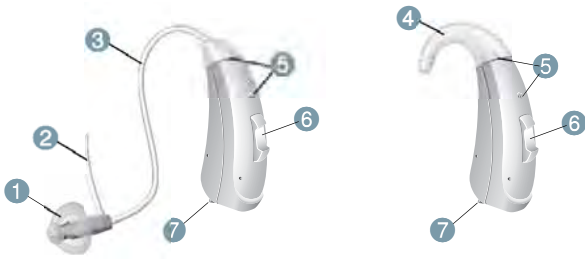


- | | |
|--------------------------|-----------------------|
| 1 Ear piece (LifeTip) | 5 Microphone openings |
| 2 Concha lock (optional) | 6 Rocker switch |
| 3 Tube (LifeTube) | 7 Battery compartment |
| 4 Earhook | |

Insertion/Removal of Battery

Remove the protective foil from the new battery. Wait for two minutes before inserting to activate the battery.

Open the battery compartment fully using your finger. Pull out the dead battery. Insert the new battery with the "+" symbol facing up. Carefully close the battery door.

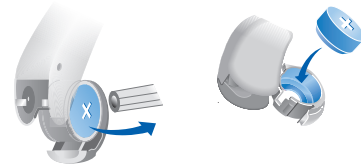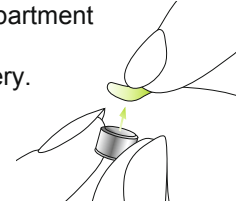

Turning the hearing aid ON/OFF

To turn the hearing aid on, close the battery compartment.

Open the battery compartment to turn off the hearing aid.



Identifying the hearing aid for the right and left ear


Side indicator:
Right = Red
Left = Blue



Rocker Switch

Short press: Volume Louder
Long Press >3 seconds: Program Up

Short press: Volume Down
Long Press >3 seconds: Program Down

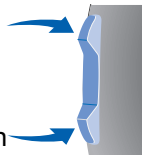

Hearing Programs

Program 1: Universal

The best choice for most listening situations. It is always active when you first switch on your hearing instrument.

Program 2: Noisy Environment

For crowded places with lots of background noise.

Program 3: Music

For listening to music.

Program 4: Phone Acoustically

For phone calls with standard phones.

Low Battery Indicator

When the battery is low the sound will become weaker or you will hear a double beep every few minutes until the battery dies.

touchControl app

Did you know you can control your hearing aids with the touchControl app?

First download the app, which is available for Android and iOS. After installation, open the touchControl app and scan the QR code. Now you are ready to use the app!



Turn your smartphone into a discreet remote control for your hearing aids. The touchControl App™ lets you enjoy a high level of handling comfort right at your fingertips.



Program and volume changes
Adapt to changing listening situations by making easy and discreet adjustments.

App download.

App-solutely easy to get!

Easy to use and easy to get: Simply download the touchControl App to your Android or iOS smartphone on the Google Play Store or Apple App Store. The best part is the download is free!

Step 1



Make sure you have a Google Play Store account for your Android phone or an Apple App Store account for your iPhone.

Step 2



Log into your Google Play Store or Apple App Store account with your Android or iOS smartphone. Find the touchControl App.

Step 3



Follow the instructions to install the app on your smartphone.

Step 4



Discover how easy it is to control your hearing instruments by simply trying your new app.